

Disposable Circumcision Anastomat

Brand new 2.0

Product Introduction

ZSR second generation disposable circumcision anastomat is a surgical instrument for circumcision and suturing Surgery on male urology . It is more time-saving more labour-saving more safer and more convenient.

The Benefit Of Circumcision



- Phimosis and redundant prepuce:
Clinical interpretation: Both phimosis and redundant prepuce refer to the foreskin covers the whole glans penis. The difference is that redundant prepuce refers to foreskin cover glans penis. Thus glans penis can not completely exposed; while phimosis refers to foreskin can't turn up and down, and the mouth of the foreskin is very small.
- To reduce occurrence probability of penis inflammation and other diseases, And to improve the quality of semen.
 - To reduce constraints and promote secondary development of penis.
 - Prolong sexual life duration and improve the quality
 - Effectively reduce microorganism breeding to make the life more healthy.
 - Reduce risk factors of reciprocal infection between couple

Several Utility Model patents and patents for innovation



New Upgrade-ZSR Second Generation Circumcision Device



Large visual operation window

The position of prepuce can be adjusted at any time.

Visible marking line

The tightness is under control.



Exclusive handle locking device, It can be locked by hitting the handle, without long pressing.


New moment arm of force structure

Super long and labor-saving handle



Metal integrated concave seat and super large frenulum observation window, make the operation more stable.

More stable to operate
Better suture

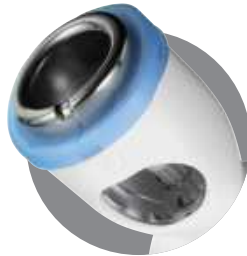

New special suturing nail for children

The suture space is smaller, and the suture is tighter.


Exclusive device to fix and protect with the frenulum

Fully protected with the frenulum
Professional guidance and safety guarantee

Operation Instructions




1 Select the appropriate size, and then put the cup inside the prepuce. (If the prepuce is adherent, shall be separated from the penis. If it is a phimosis, the preputial orifice shall be expanded or cut open with a scissors.)

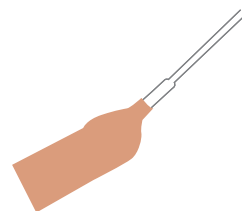


2 Make the prepuce turn out and warp the outside of cup and cover the glans. Cup frenulum-protective device catch the lace-up part.




3 Gently remove the nail-cabin protective cap, and slowly insert the screw of well-fixed cup into the center hole at the main body of anastomat. Then, turn the adjusting knob clockwise to make the marking line in the middle of the locking window.

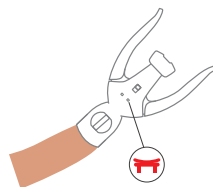

4 The prepuce cutting site can be adjusted through the observation window. After the adjustment is in place, make sure if the place where the prepuce will be cut is right or not and if the prepuce is loose or not. After everything is confirmed without any problem, remove the safety pin.

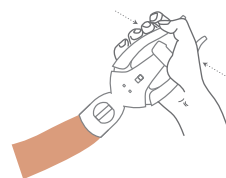

5 Fast hitting the handle and holding the flexible handgrip with strength power to trigger the anastomat. After the handle locks automatically for 15 seconds, push the lock button down to release the handle. (To prevent bleeding, immediately use dry gauze to press the operative site to stop bleeding for 2 minutes.)

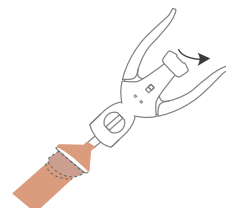

6 Loosen the flexible handgrip. Unscrew the adjusting knob, anticlockwise and slowly and observe the affected part. Slowly unscrew and back out the cup as well as the removed prepuce, and observe if the prepuce is stuck or not. If the prepuce is stuck, slightly press it with fingers or scissors, and then the prepuce will be separated naturally.

Precautions for use and Postoperative care

- 01 This product is for single use only; Please don't repeatedly disinfect it for use.
- 02 In principle, choose the bigger size, if both sizes are fine for patients.
- 03 It is forbidden to have sexual intercourse within one month after operation.
- 04 In 3 days after operation, the patient shall go back to the hospital for examination. Iodine can be used for disinfecting the wound and keeping the wound dry to prevent infection.
- 05 After the operation, patient shall follow doctor's advice, coordinate with postoperative care and use drugs. Before postoperative recovery, it is forbidden to drink alcohol and eat seafood.
- 06 After wound healed, the suturing nails will slowly fall off in one week. Due to the different physiques, some patients will need more time to have the nails fall off. If the suturing nail doesn't fall off in 4 weeks, the patient shall go back to hospital to remove it manually.

Quality assurance




Model

First Generation	A	ZSR-DCA-12、ZSR-DCA-14、ZSR-DCA-16、ZSR-DCA-18 ZSR-DCA-22、ZSR-DCA-26、ZSR-DCA-30A、ZSR-DCA-34
Second Generation	C	ZSR-DCA-12C、ZSR-DCA-14C、ZSR-DCA-16C、ZSR-DCA-18C ZSR-DCA-22C、ZSR-DCA-26C、ZSR-DCA-30C、ZSR-DCA-34C

Specification, shape and color of the device are subject to change without notice and subject to actual product configuration sheet

Measuring Scale

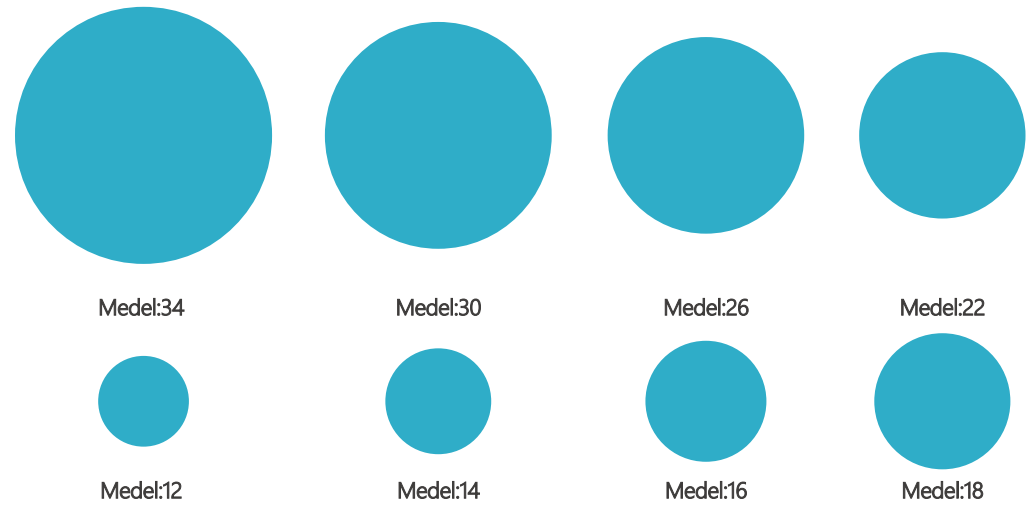

This measuring scale only use for ZSR disposable circumcision anastomat. Putting the condom into the appropriate hole while measuring, then selecting suitable disposable circumcision anastomat model to do surgical treatment when the hole fits well.